

Bio-fix

IVD In-vitro diagnostic medical device **CE CND Code: W01030705**

| Catalog number | Unit size | | | |
|------------------------|---|--------------------|-----------------------|-------------------------|
| 05-X200 | 4 x 200 ml | | | |
| Packaging | Primary container: white PVC bottle with spray and protective cap. Useful capacity 200 ml. Serigraphy resistant to water, alcohol and solvents. Secondary container: carton box. Wear, water, alcohol and solvents resistant PVC label. Scratchproof ink resistant to water and alcohol. | | | |
| Expected aim | Product for the preparation of cyto-histological samples for optical microscopy. | | | |
| Application | Spray fixative for vaginal cytology. | | | |
| Fixation technique | Spray evenly on the smear (3-4 times). Wait for 7 minutes, until the protective film is formed. The formation of a resistant and soluble film allows to best preserve the cellular material until staining. The fixed smear can be directly stained. Neverthless, removal of the waxy film in distilled water (1-2 minutes) is recommended. | | | |
| Components | Components | CAS | CE | Index |
| | Ethanol 55% | 64-17-5 | 200-578-5 | 603-002-00-5 |
| | Isopropanol 30% | 67-63-0 | 200-661-7 | 603-117-00-0 |
| | Hydrosoluble polymers 5 % | - | - | - |
| | Deionized water 10% | - | - | - |
| Warning and precaution | The product must be used exclusively by specialized technical operators. The product is classified as hazardous. Read with attention the information written on the label (dangerous symbols, risks and safety phrases). Consult always the safety data sheet where the information about the risks of the preparation, precautionary measures during use, first aid and disposal are available. Do not use if primary packaging is damaged. | | | |
| Storage | Store the preparation at room temperature. Keep the containers tightly closed. | | | |
| Stability | After the first opening, the product is usable until the expiry date, if correctly stored. Product validity: 5 years. | | | |
| Disposal | Hazardous preparation: observe all state | and local environm | ental regulations reg | garding waste disposal. |

Date of issue: December 2018

Producer: Bio-Optica Milano s.p.a.