

Cryo - Spray



CODE	DESCRIPTION	VOLUME	PACKAGING
08-SPRAY	Freezing medium spray for histology Cryo - Spray	150 ml	12 spray bottles



In vitro diagnostic – medical device



Manufacturer: Bio-Optica Milano S.p.A.
Via San Faustino 58
20134 MILANO (MI)

Date of issue: 03/09/2018
Rev. 01

Freezing medium spray for histology.

Cryo – Spray does not contain fluorinated greenhouse gases, so the Regulation (EU) 842/06 article 7 par 2 is not applicable.

Not flammable.

METHOD

The product quickly freezes tissues before cutting with cryostat and cools embedded samples before microtome cut, where the technicians have not a cold plate.

Spray directly on the sample's surface.

Specification

Specifications	Component:	HFO-1234ZE
	Colour:	Colourless
	Odour:	Characteristic
Packaging	Primary container	Spray bottle
	Label	Serigraphed surface
	Secondary container	Carton box, white colour
Warning	Storage	Store in a place where adequate ventilation is ensured, away from direct sunlight at a temperature below 50°C / 122°F, away from any combustion sources.
	Storage temperature	Store at a temperature below 50°C / 122°F.
	Stability	After the first opening, the product is usable until the expiry date, if correctly stored.
	Validity	5 years from the manufacturing date
	Product classification	The product must be used by specialized technical operators. Always refer to the safety data sheet for information on the risks of the mixture, the precautionary measures during use, and the measures first aid and in case of accidental release. Do not use if the primary container is damaged.
	Disposal	Observe all state and local environmental regulations regarding waste disposal. Bestow not used Cryo-spray to the authorized and specialized establishments, in according with the legislation about the disposal of spray bottles.

Date of issue: 03/09/2018

Rev. 01